

REMOVES SCALE, MINERAL AND ALKALINE DEPOSITS

Regular use of De-Lyme will aid your Washer Disinfector equipment as well as the cleaning process, by preventing the build up of Limescale and other mineral deposits

DIRECTIONS FOR AUTO DOSING INTO A WASHER DISINFECTOR

Light deposits for daily use:

1. Use 6ml of De-Lyme per litre of tank capacity.
2. Circulate on Acid Rinse cycle for 10 minutes.
3. Then a hot water rinse for the duration of the Disinfection Rinse.
4. Dispense where possible via an automatic dispensing system to prevent contact with the product.

Recommended use: Daily.

Heavy deposits for maintenance:

6. Use 60ml of De-Lyme per 4 litres of tank capacity.
7. Circulate on Acid Rinse cycle for 10 minutes.
8. Then a hot water rinse for at least 10 minutes.
9. Dispense where possible via an automatic dispensing system to prevent contact with the product.

Recommended use: Weekly.

DIRECTIONS FOR MANUAL DOSING INTO A WASHER DISINFECTOR

1. Use 6ml of De-Lyme per litre of tank capacity.
2. Pour product into a large Kidney Dish
3. Place on an empty carriage in the centre of the washer disinfector
4. Run washer disinfector on standard instrument cycle
5. May require several runs dependent on build up.

Recommended use: As often as desired.

CAUTION: DE-LYME CONTAINS PHOSPHORIC ACID

Irritating to eyes and skin.

In case of contact with eyes rinse immediately with plenty of water and seek medical advice.

In case of accident or if you feel unwell, seek medical advice immediately.

Available in:

Cat No 345DL 4 x 4 Litres
Cat No 345DL10L 2 x 10 Litres