

ENDOZIME AW PLUS®

Multi-Tiered™ Low Foaming Enzymatic Cleaner Specifically Designed for Manual Cleaning, Ultrasonic and Washer Disinfectors.

A unique synergistic formulation of protease, amylase, lipase, carbohydrase and proprietary enzymes that remove fat, blood, protein and carbohydrates from all surgical instruments and scopes in two minutes. At 4 ml's per litre of water, Endozime AW Plus is the most powerful enzymatic cleaner and a complete detergent with buffers and surfactants

The ultimate bacteriostatic enzymatic detergent designed as a total cleaning product for use in all endoscopic washers, washer disinfectors, ultrasonics and manual cleaning. Low foaming and neutral pH Endozime AW Plus eliminates the need for all other cleaners or detergents.

Developed for cleaning the most difficult instruments (i.e., orthopaedic, laparoscopic), Endozime AW Plus has proven to be safe for use on all instruments and scopes, even the most delicate (i.e., ophthalmic, microsurgical). Clinically tested to be Bacteriostatic, free rinsing and 100% biodegradable.

- Proprietary synergistic mixture of enzymes and non-ionic detergents
- Instantly active Constantly active
- Starts cleaning on contact
- Neutral pH (non-abrasive)
- Easy and safe to use
- Will not harm any metals, plastics, rubber or corrugated tubing
- Works in all temperatures

AVAILABLE IN

Cat No 345AP1 - 12 x 500ml

Cat No 345AP - 4 x 4 Litres

Cat No 345AP10L - 2 x 10 Litres



DIRECTIONS FOR USE

Manual Cleaning

1. Add 4 ml of Endozime AW Plus to 1 litre tepid water. Use Measuring Jug, Syringe, Pump Dispenser (For 4 Litre Bottle) Dispenses 30ml per Squirt or auto dispenser if fitted
2. Submerge instruments and scopes to be cleaned. For scopes: suction or flush through channels before soaking.
3. Soak for two minutes to remove all organic matter.
4. Rinse thoroughly with tap, distilled or sterile water.
5. Discard used solution.

Automatic Cleaning Devices

- Pour directly into tank at 4ml per litre of water or
- Add to the detergent reservoir full strength or
- Fit detergent bottle to washer disinfectant pump