

ENDOZIME AW TRIPLE PLUS® (WITH A.P.A.)

NO SCENT®

Multi-Tiered Enzymatic Detergent with Advanced Proteolytic Action and Rust Inhibitors with No Dye or Scent.

Specifically designed for customers who are susceptible to allergic reactions when using a cleaning chemicals containing dye and perfume (scent). The product can replace the regular Endozime AW Plus in automatic washers, washer disinfectors, ultrasonic and manual cleaning.

The unique formulation of protease, amylase, lipase, carbohydrase and proprietary enzymes with A.P.A. (Advanced Proteolytic Action), which removes all soil and inhibits rust. A.P.A. is the latest enzymatic breakthrough developed by Ruhof Healthcare that greatly increases protein enzyme activity to provide a faster more thorough penetration into hard to reach places on surgical instruments and scopes.

At 4ml's per litre of water, Endozime AW Plus is the most powerful enzymatic cleaner and a complete detergent with buffers and surfactants. One solution for all your cleaning problems. Clinically tested to be free rinsing and 100% biodegradable.

- Proprietary synergistic mixture of enzymes and non-ionic detergents
- Instantly active – Constantly active
- Starts cleaning on contact
- Bacteriostatic
- Easy and safe to use
- Safe for all instruments and scopes
- Neutral pH (non-abrasive)
- Low-foaming
- 100% Biodegradable
- Free rinsing
- Will not harm any metals, plastics, rubber or corrugated tubing
- **Works in all water temperatures**



DIRECTIONS FOR USE

Surgical Instruments and Scopes

1. Add 4ml of Endozime AW Plus to 1 litre of warm water.
Pump Dispenser (for 4 litre bottle) dispenses 30ml per squirt.
2. Submerge instruments and scopes to be cleaned.
For scopes: suction or flush through channels before soaking.
3. Soak for two minutes to remove all organic matter.
4. Rinse thoroughly with tap, distilled or sterile water.
5. Discard used solution.

Automatic Cleaning Devices

- Pour directly into detergent reservoir full strength, or
- Dilute 4ml per litre of water.
- Discard used solution.

Available in:

Cat No 345APANS 4 x 4 Litres



Peskett Solutions Ltd
Unit 4-5 Brampton Business Park
55 Brampton Road, Eastbourne, East Sussex, BN22 9AF
Tel: 01323 511038 Fax: 01323 503247
E Mail: info@ruhof.co.uk Web: www.ruhof.co.uk