

## DIGITAL SINK THERMOMETER



With a single cable sensor which allows the temperature of your sink to be accurately measured whilst the display remains outside. The remote probe can be mounted onto the sink wall using the suction pad supplied. The Thermometer is designed to stay on all the time, therefore there is no off button for the unit.

**Instrument Operation:** - The instrument will turn on once the battery is installed. **Please note:** this product cannot be placed in the water. Place the probe within the water using the sucker provided. The instrument reading may take a few seconds to stabilise. The probe will measure the temperature of the water. If the screen is displaying the word "ROOM", press the probe button for the correct setting.

**Battery Replacement** – The battery will need to be replaced when the instrument screen dims. Replace the battery with a 3volt lithium CR2032 coin cell. The battery is located under the battery cover on the back of the instrument.

### Optional Extras

**Temperature Alarm Setting - Probe** – To set the audible alarm, press and hold the ALARM SET button, the high alarm temperature will flash in the display. To set the high alarm, press or hold the red button on the back of the unit to adjust the temperature. To set the low alarm press the ALARM SET button again and press or hold the red button to adjust the temperature. Press the ALARM SET button to confirm. To turn the alarm on press the ALARM SET button and "AL" will be shown in the display. The audible alarm will only sound when the temperature had been exceeded for 5 Seconds and again every 55 seconds to reset the minimum and maximum temperature memory.

**Max/Min** - Press the MAX/MIN button to display the maximum reading (MAX Displayed). Press the MAX/MIN button again to display the minimum reading (MIN Displayed) Press MAX/MIN button again to return to the normal reading.

**Please note** - Max/Min reading are not saved so removal of the battery will clear the max/min readings.