

RUHOF ENDO-BAG®


SCOPE SMALL ACCESSORY CLEANING BAG

The Endo-Bag is perfect for keeping Scope buttons and valves with the scope in the washer disinfector during the cleaning and disinfection process.

Perfect for use in Endoscopy Suites, Operating Theatres, Sterile Services, Outpatient Surgery, and clinics where small items require cleaning in an automated process.

Benefits

- Keeps the scope and its accessories together during the cleaning process.
- Protects the accessories during the cleaning process.
- Assists in the traceability of the scope.
- Can be used for any small item in any type of washer disinfector.
- Designed to withstand High Level Disinfectant.
- Made of strong, non-linting nylon.

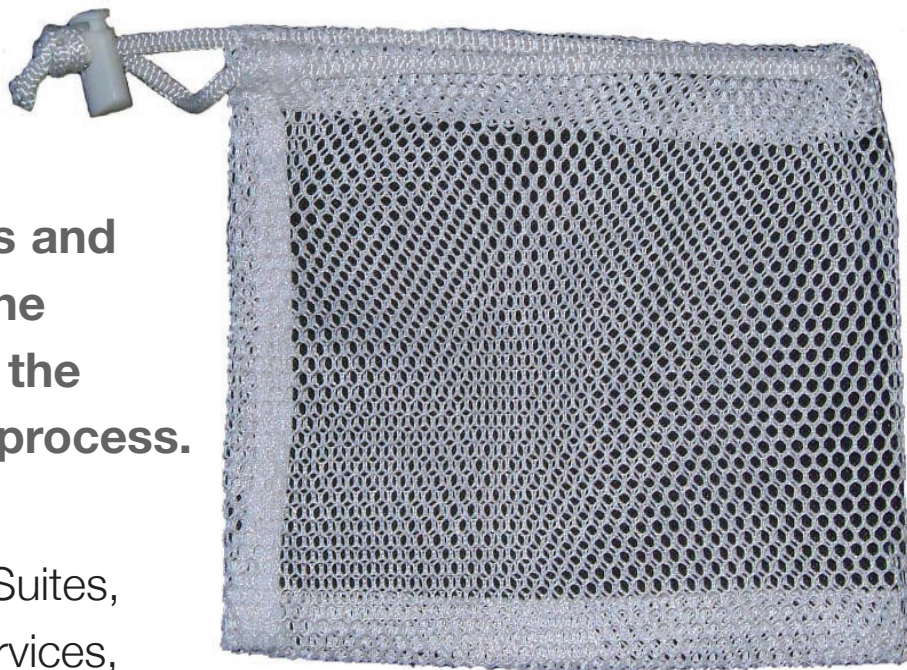

AVAILABLE IN

Cat No 345EDB
Unit of issue - 50

DIRECTIONS FOR USE

- 1) Place the small items for cleaning in open bag.
- 2) Holding the cord, gather the top of the bag to close the top.
- 3) Press the button on the toggle and pull it along to the closed bag to hold it in place.
- 4) Place the loop formed by the closure, around the control knobs of the Scope.

PLEASE NOTE:

After prolonged use the Endo-Bag may require replacing if there is any sign of deterioration of the bag fibres.