

ENDOZIME®

eXtreme Power

SUPER CONCENTRATE

USE LESS
Clean More

Multi-Tiered Enzymatic Detergent with Advanced Proteolytic Action and Rust Inhibitors.

Designed for use in all Endoscope Washers, Washer Disinfectors, Ultrasonic Washers and For Manual Cleaning

Endozime® Extreme Power combines the power of a super-concentrate with the performance of Ruhof's proven multi-enzyme formula to create the ultimate surgical instrument and scope cleaner. This unique combination gives Endozime® Extreme Power its superior cleaning abilities to deliver fast and thorough soil contamination removal. Enhanced with APA (Advanced Proteolytic Action) that greatly increases protease enzyme activity. Endozime® Extreme Power penetrates into the hardest-to-reach channels of scopes and cannulated instruments, instantly and actively breaking down gross contaminants upon contact. Endozime® Extreme Power contains biological additives that speed the process of liquification and solubilization, facilitating enzymatic action and contributing to the product's overall effectiveness.

- Super-Concentrate: Dilution from 1/4 oz per gallon (2ml per Litre)
- Effective in all water temperatures (Including cold water)
- No foam formula
- Removes all soil contamination. Starts cleaning on contact
- Available in smaller bottles. Easier to carry, store and pour.
- Compatible with all medical device materials.
- Neutral pH, Free rinsing and 100% Biodegradable
- Inhibits rust

Environmental and cost benefits:

Less detergent usage
100 % biodegradable
Less plastic bottle and packaging waste
Reduction in energy usage



DIRECTIONS FOR USE:

Automatic Cleaning Devices:

Pour 2 ml per litre directly into detergent reservoir full strength, or calibrate the washer dilution at 2 ml per litre of water.

Follow manufacture's recommendations for correct dosing option

Manual Cleaning

1. Add 2ml per litre of Endozime Extreme Power (*Works in all water temperatures*)
2. Submerge instruments and scopes to be cleaned. (*For scopes: suction or flush through channels before soaking*)
3. Soak for two minutes to remove all organic matter. (*Heavy soil may require longer time*)
4. Rinse thoroughly with tap, distilled or sterile water.
5. Discard used solution.



Peskett Solutions Ltd
Unit 4-5 Brampton Business Park
55 Brampton Road, Eastbourne, East Sussex, BN22 9AF

Tel: 01323 511038 Fax: 01323 503247
e mail: info@ruhof.co.uk web: www.ruhof.co.uk