

# F\*ACT\*S\*

## FOAMING AUTOCLAVE CLEANING TREATMENT SYSTEM EFFICIENT AND QUICK FOAMING REVITALISING SOLUTION

**F.A.C.T.S Will Remove All Rust, Spots, Stains And Corrosion, As Well As Scale, Alkaline And Mineral Deposits On The Inside Walls Of The Autoclave And From Stainless Steel Carts And Trays.**

- Foaming spray remains on the walls longer, allowing for better penetration and more effective cleaning.
- It simultaneously eliminates all noxious and unpleasant odours.
- F.A.C.T.S is free rinsing and leaves no residue.
- Non-aerosol pump is environmentally safe.
- Repeat process as often as desired.
- Regular use of F.A.C.T.S. will prolong the life of your stainless steel equipment.



### DIRECTIONS FOR USE:

#### For Autoclave And Washer Disinfector Cleaning

1. Autoclave walls must be cool.
2. Spray F.A.C.T.S. generously on inside of the chamber.
3. Allow to remain wet for 15-30 minutes.
4. Brushing with non-metallic brush will speed the action.
5. Use of gloves is recommended.
6. Rinse walls with copious amounts of water.

#### For Stainless Steel Carts, Trays And Surfaces

1. Spray F.A.C.T.S. generously on surfaces.
2. Allow to remain wet for 15-30 minutes.
3. Brushing with non-metallic brush will speed the action.
4. Use of gloves is recommended.
5. Rinse the surface with copious amounts of water.

**CAUTION: DO NOT USE ON CHROME OR SILVER PLATED EQUIPMENT**

### Available in:

Cat No 345FCT 650ml Spray Bottle

**R** **RUHOF**   
THE EXPERTS IN A BOTTLE

Peskett Solutions Ltd  
Unit 4-5 Brampton Business Park  
55 Brampton Road, Eastbourne, East Sussex, BN22 9AF  
Tel: 01323 511038 Fax: 01323 503247  
E Mail: info@ruhof.co.uk Web: www.ruhof.co.uk