

# PREMIXSLIP SPRAY & DROPPER

The only premixed instrument lubricant and rust inhibitor clinically tested to be thoroughly steam penetrable and steam sterilisable.

Premixslip is a non-oily, non-sticky, non-toxic, silicone free lubricant for surgical instruments. It prevents rusting, staining, spotting and tightness of surgical instruments. Premixslip reduces costly repair and replacement of surgical instruments. Premixslip should be used on a regular basis on all surgical instruments with box joints to prevent tightening and sticking.

The dropper is ideal for small instruments and box joints.



## Water Soluble v's Oil Based

The addition of oil-based compounds to the cleaning process is wrong in principle. They deliberately cause contamination over the entire cleaned surface. If they are to be used the water-soluble type should be used. Mineral oils have poor biocompatibility and may inhibit the penetration of steam or sterilant gases on terminally sterilized product.

Lubrication should only be applied to those areas where it is required during the inspection/packing process after thorough cleaning of the instrument.

### AVAILABLE IN

Cat No 345PMX1	12 x 500ml	With Sprayer
Cat No 345PMX1D	20 x 250ml	With Dropper

## DIRECTIONS FOR USE

Use as is –

**Do Not Dilute – Shake gently prior to use**

For Spray Applications:

1. Restrict the use of the lubricant to the area of the instrument requiring lubrication using the sprayer or dropper.
2. Allow the Premixslip to penetrate the joint for about 30 seconds
3. Test the action of the lubricated joint to ensure penetration
4. For exceptionally stiff instruments re-apply and actively open and shut the instrument, until loosened.
5. If the lubricant dislodges dried contaminants from the joint. The instrument will require re-cleaning.
6. Seized instruments may require treatment by immersion in a bath of Premixslip.

**R** **RUHOF**   
THE EXPERTS IN A BOTTLE

Peskett Solutions Ltd  
Unit 4-5 Brampton Business Park  
55 Brampton Road, Eastbourne, East Sussex, BN22 9AF  
Tel: 01323 511038 Fax: 01323 503247  
E Mail: info@ruhof.co.uk Web: www.ruhof.co.uk