

PREPZYME®

MULTI-TIERED ENZYMATIC FOAM SPRAY

PREPZYME® is ideal for keeping soiled instruments, scopes and biopsy forceps moist, preventing bio-burden from adhering to the surface prior to decontamination.

The multi-tiered enzymatic foaming spray covers the instruments and begins breaking down blood, fat, protein and carbohydrates before they arrive in the Sterile Services Department. The Prepzyme contains a rust inhibitor that protects the instruments during transportation.

Perfect for use in Operating Theatres, Endoscopy Suites, Outpatient Surgery, Dental and other off-site departments and clinics where instrument and scope cleaning is not immediately available. No more messy spills. Stops splashing during transportation. Helps reduce cleaning time.

The applicator allows you to administer Prepzyme directly into the lumen of most cannulated instrument after use



AVAILABLE IN

Cat No 345PRP

6 x 650ml Bottles

- Safe for all instruments and scopes, flexible and rigid, extends usable lifetime
- Neutral pH (non-abrasive), 100% biodegradable
- Free rinsing
- Will not harm any metals, plastics, rubber or corrugated tubing
- Specially designed non-aerosol sprayer is environmentally friendly.

DIRECTIONS FOR USE

- 1) Spray directly on soiled instruments and scopes immediately after use.
- 2) Allow foam to stay on instruments and scopes until ready for cleaning. Apply more as needed to keep moist.
- 3) Rinse instruments thoroughly before processing in a Washer Disinfector.

RUHOF 
THE EXPERTS IN A BOTTLE

Peskett Solutions Ltd
Unit 4-5 Brampton Business Park
55 Brampton Road, Eastbourne, East Sussex, BN22 9AF
Tel: 01323 511038 Fax: 01323 503247
E Mail: info@ruhof.co.uk Web: www.ruhof.co.uk