

PREPZYME®

MULTI-TIERED ENZYMATIC FOAM SPRAY

PREPZYME® is ideal for keeping soiled instruments, scopes and biopsy forceps moist, preventing bio-burden from adhering to the surface prior to decontamination.

The multi-tiered enzymatic foaming spray covers the instruments and begins breaking down blood, fat, protein and carbohydrates before they arrive in the Sterile Services Department. The Prepzyme contains a rust inhibitor that protects the instruments during transportation.

Perfect for use in Operating Theatres, Endoscopy Suites, Outpatient Surgery, Dental and other off-site departments and clinics where instrument and scope cleaning is not immediately available. No more messy spills. Stops splashing during transportation. Helps reduce cleaning time.



- Safe for all instruments and scopes, flexible and rigid, extends usable lifetime
- Neutral pH (non-abrasive), 100% biodegradable
- Free rinsing
- Will not harm any metals, plastics, rubber or corrugated tubing
- Specially designed non-aerosol sprayer is environmentally friendly.

DIRECTIONS FOR USE

- 1) Spray directly on soiled instruments and scopes immediately after use.
- 2) Allow foam to stay on instruments and scopes until ready for cleaning. Apply more as needed to keep moist.
- 3) Rinse instruments thoroughly before processing in a Washer Disinfector.

Available in

NHS Supply Chain Code:

Cat No 345PRP

6 x 650ml Bottles

FAL744

RUHOF 
THE EXPERTS IN A BOTTLE

Peskett Solutions Ltd
Unit 4 Brampton Business Park
55 Brampton Road, Eastbourne, East Sussex, BN22 9AF
Tel: 01323 511038 Fax: 01323 503247
E Mail: info@ruhof.co.uk Web: www.ruhof.co.uk